

QUASAR™

ISOCENTER CUBE

Phantom

A simple yet powerful daily QA test to verify IGRT setup accuracy

The QUASAR™ IsoCenter Cube is designed to implement a modified Winston-Lutz test for IGRT accuracy required for optimal treatment delivery. This system is also used for TG-142 quality assurance compliance for isocenter and laser alignment tests.



End-to-end positioning accuracy from planning CT to clinical IGRT treatment setup is achieved with 0.1 mm precision and is accomplished in minutes. The test is performed by aligning the QUASAR™ IsoCenter Cube Phantom using the cross-hairs etched on each side with your IGRT system,

KEY FEATURES

- ▶ Compatible with TG142 tests
- ▶ Check various beam shaping devices
- ▶ Analyze EPID radiation field exposures
- ▶ Quick and simple Winston-Lutz testing for IGRT accuracy
- ▶ Report left/right, superior/inferior and anterior/posterior errors at configurable gantry, collimator and table angles



Above: The acrylic 5 x 5 x 5 cm QUASAR™ IsoCenter Cube Phantom used for measuring isocenter accuracy

then capturing radiation field exposures with film or EPID. The test results are used to determine end-to-end accuracy for frameless SRS, SBRT, and IGRT treatments.

The QUASAR™ IsoCenter Cube Phantom features a seamless acrylic 5 x 5 x 5 cm unibody enclosure containing one 6 mm diameter Tungsten Sphere located at the central axis of the main cross-hairs and used for measuring isocenter accuracy. Displayed on 3 faces are 3 mm offset laser alignment marks, which are used for couch position accuracy. Test end-to-end IGRT positioning accuracy for clinical treatment setups with 0.1 mm precision in less than 5 minutes.

- ▶ Perform test by aligning the phantom containing radiopaque sphere with IGRT system, then capturing radiation field exposures with film or EPID
- ▶ Use the test results to determine end-to-end accuracy for frameless SRS, SBRT, and IGRT treatments




Accuracy. Confidence.™


WINSTON-LUTZ ANALYSIS SOFTWARE

Analyze EPID radiation field exposures with the automated Analysis Software. Check various beam shaping devices, report left/right, superior/inferior and anterior/posterior errors at configurable gantry, collimator and table angles.


Ability to test IGRT accuracy with a minimum of two gantry angle images. The user sets the warning and fail levels, and can visually see pass (green), warning (yellow), and fail (red) results.

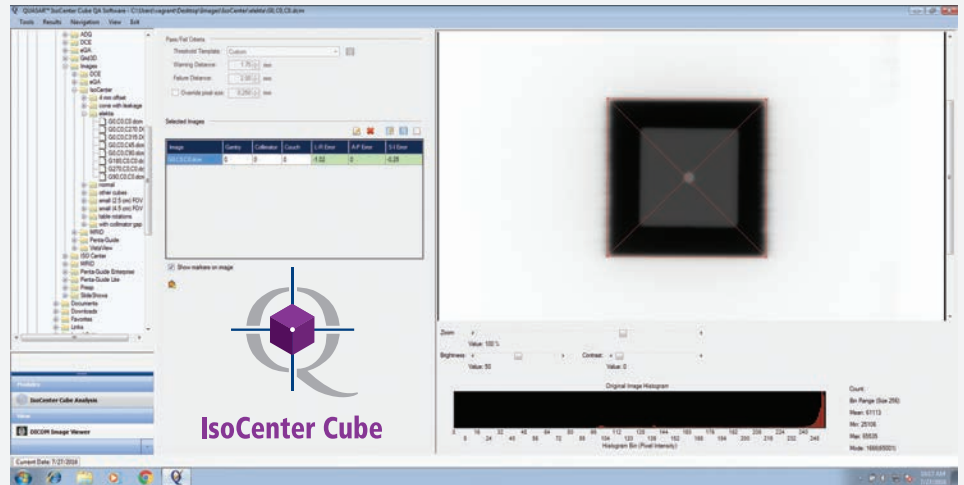

Above: Screenshot of the QUASAR™ IsoCenter Cube analysis software displaying pass (green) results

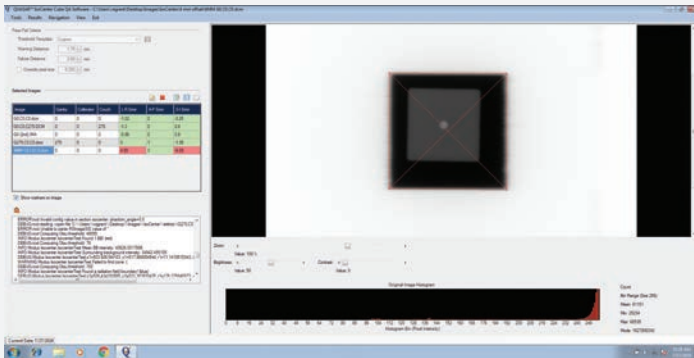

Above: Screenshot of the QUASAR™ IsoCenter Cube analysis software displaying fail (red) results

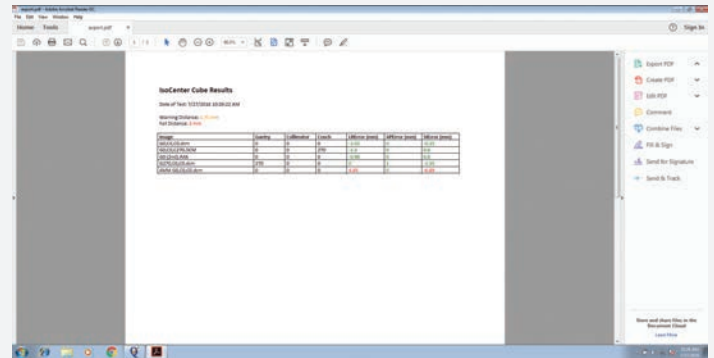

Above: Screenshot of the QUASAR™ IsoCenter Cube analysis software displaying pass (green), fail (red) results

MINIMUM TECHNICAL REQUIREMENTS

- ▶ **Operating System:** Windows 7 or higher
- ▶ **Processor:** Intel Core™ i7 or better
- ▶ **Hard Disk Space:** 5 GB minimum (64-bit)
- ▶ **RAM:** 1 gigabyte (GB) or more
- ▶ **Screen Resolution:** 1280 x 1024 pixels or higher
- ▶ **Graphics Card:** Intel HD Integrated Graphics or better

SPECIFICATIONS

- ▶ Acrylic cube; 5 x 5 x 5 cm with no-slip rubber base
- ▶ Tungsten sphere at isocenter 6mm diameter
- ▶ Laser alignment marks on cube
- ▶ Offset laser alignment marks on 3 faces

ORDERING INFORMATION

- 100-1016 QUASAR™ IsoCenter Cube Package
 - ▶ 1-Phantom
 - ▶ 1-Software
 - ▶ User's guide
- 100-1016-001 Additional Phantom

© 2018 Modus Medical Devices Inc. All Rights Reserved. Specifications subject to change without notice. Modus QA is not responsible for errors or omissions. Modus QA makes no warranties or commitments concerning the availability of future products or versions that may be planned or under development. PDS#100-1016 REV#06/25

Modus QA

1570 North Routledge Park,
London, Ontario Canada N6H 5L6

[e] info@modusQA.com
[w] www.modusQA.com

Toll Free: +1 (866) 862-9682 (North America)
Phone: +1 (519) 438-2409
Fax: +1 (519) 643-0127

GET SOCIAL



modusQA.com